## **URGENT: FIELD SAFETY NOTICE**

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#### **Batteries Supplied by CSB and Used with Plum Infusion Systems**

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ICU Medical identified a potential issue with batteries used in Plum infusion systems. The attached Urgent FSN letter describes the issue and the actions for users to take.

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#### **1. Q What is the issue?**

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ICU Medical identified a potential issue with replacement batteries used in Plum Infusion Systems. Due to a manufacturing defect from the battery supplier, affected batteries may experience a loss of capacity earlier in the battery lifecycle than expected and overall battery runtime may decrease earlier in the battery lifecycle than expected.

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If the pump is running on battery power, the user may not have sufficient time to plug the pump into AC power after the Low Battery alarm is activated, which may result in an interruption of therapy. An interruption of therapy may potentially lead to serious patient injury or death, depending on the clinical situation and the type of medication being administered.

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**3. Q What products are affected by this notice?**

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**4. Q Has ICU Medical corrected this issue?**

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Yes. ICU Medical worked with CSB to identify and correct the manufacturing defect that led to this issue.

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**5. Q Are all Plum batteries affected by this field notice?**

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**6. Q How do I know if I have affected batteries?**

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All affected batteries have the CSB logo.

Also the first two characters on the following label indicate the year the battery was manufactured. If the first two characters are 22 or lower, this issue potentially affects the battery.

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**7. Q What do I do with the affected batteries in my stock?**

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ICU Medical will replace all replacement batteries you have in your possession free of charge. ICU Medical is working to secure as many batteries as possible from our manufacturer as quickly as possible to replace all batteries with all customers.

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**8. Q What should I do until I get my allotment of new batteries?**

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Your Plum A+ and Plum 360 pumps are safe to use. Please identify all affected batteries in your possession and ensure all users or potential users of these pumps are immediately made aware of this notification and proposed mitigations.

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**9. Q Can non-ICU Medical batteries be used with Plum A+ or Plum 360 pumps?**

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No. ICU Medical cannot validate the quality of any alternative battery and will not reimburse any purchases for non-ICU Medical batteries.

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**10. Q Do I need to stop using my Plum 360 and/or Plum A+ pumps?**

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No. Whenever possible, keep the pump plugged into AC power. Before disconnecting the pump from AC power, e.g., to transport a patient, please ensure that the battery is fully charged. Closely monitor the Battery Status Indicator while the pump is disconnected from AC power to help ensure there is sufficient battery capacity to power the pump. Additionally, have a backup pump available when infusing critical medications.

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**11. Q Will ICU Medical come on-site and replace all batteries?**

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No. ICU Medical will not come on-site to remove/replace all batteries. ICU Medical will ship replacement batteries to you as quickly as our supply allows.

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**12. Q Can I return my pumps to ICU Medical to replace my device batteries?**

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No. There is no need to return your Plum A+ or Plum 360 as the pumps are safe to use. ICU Medical will send replacement batteries to you as quickly as our supply allows.

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**13. Q Has there been any patient harm due to this issue?**

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