## URGENT: FIELD SAFETY NOTICE

**\*\*UPDATED\*\***

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**\*\*ACTUALIZADO\*\***

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#### Batteries Supplied by CSB and Used with Plum Infusion Systems

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ICU Medical identified a potential issue with batteries used in Plum infusion systems. The attached **UPDATED** - Urgent FSN letter describes the issue and the actions for users to take.

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#### 1. Q What is the issue?

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ICU Medical identified a potential issue with replacement batteries used in Plum Infusion Systems. Due to a manufacturing defect from the battery supplier, affected batteries may experience a loss of capacity earlier in the battery lifecycle than expected and overall battery runtime may decrease earlier in the battery lifecycle than expected.

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If the pump is running on battery power, the user may not have sufficient time to plug the pump into AC power after the Low Battery alarm is activated, which may result in an interruption of therapy. An interruption of therapy may potentially lead to serious patient injury or death, depending on the clinical situation and the type of medication being administered.

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**3. Q What products are affected by this notice?**

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**4. Q Has ICU Medical corrected this issue?**

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ICU Medical is working with our battery supplier to resolve this issue fully. We will communicate further updates when new information becomes available.

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**5. Q Are all Plum batteries affected by this field notice?**

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**6. Q How do I know if I have affected batteries?**

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All affected batteries have the CSB logo.

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**7. Q What do I do with the affected batteries in my stock?**

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Potentially affected CSB batteries that you have in stock can continue to be used as replacement batteries for the Plum Infusion System provided you take the precautions described in the FSN notice and in Q10 below. The Frequency of Occurrence for this loss of capacity issue for Plum 360 per ICU Medical Risk Assessment is 0.19%.

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**8. Q Will ICU Medical replace all potentially affected CSB batteries?**

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No. Potentially affected CSB batteries are safe to use following the precautionary steps in Q10 below. Per Q4 above, Customers will be informed when this issue has been resolved.

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**9. Q Can non-ICU Medical batteries be used with Plum A+ or Plum 360 pumps?**

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No. ICU Medical cannot validate the quality of any alternative battery and will not reimburse any purchases for non-ICU Medical batteries.

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**10. Q Do I need to stop using my Plum 360 and/or Plum A+ pumps?**

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No. Whenever possible, keep the pump plugged into AC power. Before disconnecting the pump from AC power, e.g., to transport a patient, please ensure that the battery is fully charged. Closely monitor the Battery Status Indicator while the pump is disconnected from AC power to help ensure there is sufficient battery capacity to power the pump. Additionally, have a backup pump available when infusing critical medications.

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**11. Q Has there been any patient harm due to this issue?**

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**12. Q Why are there additional Plum A+ pump list numbers in the product table?**

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The pump list numbers mentioned in the table is intended to cover ALL versions of the pump. Not all list numbers are applicable to every country

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The Pump A+3 is the 'triple' version of the Plum A+ device. Collectively, both can be referred to as Plum A+. Both Plum A+ and Plum A+3 pumps are affected by this notice.

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**14. Q Is it ok to reset the battery in Biomed Mode without replacing the battery?**

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No, a pump that has alarmed and indicated the battery needs to be replaced requires a battery replacement to correctly resolve this alarm

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